



VIVO 205

Use 675A battery
Height: 1 1/2"

Volume Control
On / Off Switch


With stable and high performance, VIVO205 offers a better choice for your cost-sensitive clients.

VIVO205


- * Middle power BTE
- * Mini size
- * Volume control
- * N-H switch



Testing diagram

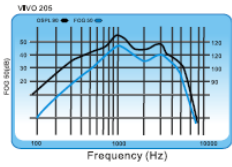

Fitting range

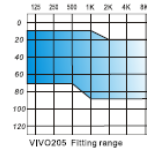

Technical Data

Technical Data	VIVO205
Peak Value	125 dB
Output at 1600 Hz	120 dB
Maximum Gain	45 dB
Gain at 1600 Hz	40 dB
Battery Current	1.3mA
Equivalent Input Noise	27dB
Total Harmonic Distortion	0.3%

IEC-118-7/94 (2cc-coupler)



سماعة خلف الأذن - موديل فيفو ٢٠٥

خصائص السماعة :

- زر خاص للتشغيل والإيقاف (I / O) .
- عجلة تحكم لرفع و خفض درجة الصوت (١ - ٤ درجات) .
- سماعة صغيرة الحجم (مدمجة) - بصوت واضح و نقي .



مرفقات مع السماعة :

- عدد (٢) تيوب - عدد (٤) قوالب للأذن مقاسات متعددة
- عدد (١) بطارية موديل (٦٧٥ - ١.٤ فولت)
- عدد (١) منديل للمسح - عدد (١) فرشاة للتنظيف
- علبة حافظة للسماعة - كرت ضمان - كتيب إرشادات

طريقة الإستخدام :

- قبل البدء باستخدام السماعة قم بتنظيف الأذن من الصمغ باستخدام الأعواد القطنية .
- إسحب حجيرة البطارية للأسفل . أدخل البطارية في القالب المخصص بحيث يكون الطرف الموجب (+) الى الأعلى . قم بغلق الغطاء .
- قم بتوصيل التيوب مع قالب الأذن (إختار المقاس المناسب)
- قم بتحديد طول التيوب . ثم قم بقص الطول الزائد .
- قم بتشغيل الجهاز (I / O) . حدد مستوى الصوت المناسب بتحريك عجلة التحكم . قم بوضع السماعة على الأذن .
- أزل السماعة عن الأذن و قم بإطفائها عند النوم .
- (للتنظيف والحفظ - قم باستخدام المنديل والفرشاة المرفقة)
- (إحذر ملامسة الماء والعرق الشديد للسماعة مباشرة)

